LUXAMED®

LUXAMED® LuxaScope Sonus und Sonus Flex Stethoskope

LUXAMED® LuxaScope Sonus and Sonus Flex Stethoscopes

LUXAScope Sonus et Sonus Flex Stethoscopes LUXAMED®

LUXAMED® LuxaScope Sonus e Sonus Flex Stetoscopi

LUXAMED® LuxaScope Sonus y Sonus Flex Estetoscopios

Stetoskopy LUXAMED® LuxaScope Sonus i Sonus Flex

LUXAMED® LuxaScope Sonus e Sonus Flex Estetoscópios

Deutsch	2
English.....	6
Français.....	10
Italiano	14
Español.....	18
Polski	22
Português	26

LUXAMED®

LUXAMED GmbH & Co. KG *

Daniel-Weil-Str.3 * 89143 Blaubeuren * Germany

Telefon: +49(0)7344 92905 0 * Fax: +49(0)7344 92905 10

E-Mail: info@luxamed.de * www.luxamed.de

- DEUTSCH -

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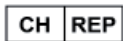
- ENGLISH -

Before using the LUXAMED® LuxaScope Sonus and Sonus Flex stethoscopes and their accessories, read these instructions carefully and keep them for later reading.

LUXAMED® LuxaScope Sonus and Sonus Flex Stethoscopes

Purpose:

The LUXAMED® LuxaScope Sonus stethoscopes are used to auscultate a patient to listen to audible organ activity. Furthermore, the stethoscopes can be used to support blood pressure measurement according to Korotkow. The instruments may only be used by medically trained personnel.

LUXAMED® stethoscopes are latex-free.

Use the device as intended, only for the approved purpose.

Contraindications:

none

Warnings:

Before each application, make sure that SOFTBUDS ear olives are neatly attached to the temple. NEVER use without SOFTBUDS ear olives.

Adjust the temples ergonomically according to your needs to achieve optimal hearing quality.

Make sure that the SOFTBUDS ear olives do not protrude too far into the ear canal. This can lead to injuries.

Keep the SOFTBUDS ear olives and temples clean from impurities so as not to deteriorate the hearing quality.

Use the stethoscopes only with Luxamed spare parts and accessories.

Any serious incidents involving the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Application:

The stethoscopes are used by medical personnel to listen to physiological sounds and sounds. In the blood pressure measurement according to Korotkov, stethoscopes are used to determine the systolic and diastolic value.

A stethoscope consists of chest piece, tube system with ear hook and attached SOFT-BUDS ear olives at their ends.

To adapt the ear hooks to your physiological conditions, you can bend the two temples apart or squeeze them to create more pressure between SOFTBUDS ear olives and ears. Proceed carefully to avoid overstretching.

The stethoscopes have 2 sides, membrane side and funnel side. These can be selected depending on the frequency range. To do this, hold the chest piece in one hand and the hose attachment in the other. Now turn the chest piece 180° until it snaps into place. If the sound inlet hole on the funnel side is closed, the membrane side is active. If the funnel side is active, you can use the funnel or funnel side. listen to the smaller side of the membrane.

Stethoscope variants:

LuxaScope Sonus: Stainless steel stethoscope with switching for high and low frequency tones, available in different colors and 2 sizes (Adult/Pediatric, Pediatric/Neonatal)

LuxaScope Sonus Flex Stethoscopes: Stethoscope with switching for high and low frequency tones, available in different colors and 1 size (Adult), interchangeable clip diaphragm and interchangeable clip funnel ring by simply attaching and unplugging.

LuxaScope Sonus FLAT Flex Stethoscopes: Flat head stethoscope, available in different colors and 1 size (Adult), interchangeable interchangeable clip diaphragms by simply attaching and unplugging.

Cleaning:

The stethoscope must be wiped with a damp cloth (e.B. slightly alkaline or pH-neutral detergents) after each application.

Disinfection:

Wipe disinfection can be used to disinfect or disinfect the outer surfaces of the stethoscope. For this purpose, disinfectants can be used that are approved for medical devices made of plastic or rubber. Spray and immersion disinfection are not permitted. LuxaScope Sonus Flex stethoscopes do not necessarily need to be disinfected. The interchangeable clip diaphragm or clip funnel ring must be disinfected or renewed after use as described above.

Sterilization:

Sterilization is not permitted.

Disposal:

The product and its accessories can be disposed of as household waste.

Storage/transport conditions and operating conditions:

Ambient temperature between -10° and +40°C.

Rel. Humidity between 10% and 90%.

Symbol key

	Manufacturer
	Date of manufacture
	Observe the instructions for use
	Warnings
	Compliant with EU regulations
	The device is a medical device
	Compliant with UK regulations

Warranty conditions

For this product we provide 2 years factory warranty from the date of purchase. The date of purchase must be proven by invoice. Within the warranty period, defects resulting from material or manufacturing defects will be remedied free of charge. The warranty service does not extend the warranty period to the entire device, but only to the replaced components. Excluded from the warranty is wear and tear due to wear and tear, transport damage and all damage caused by improper handling (e.B. non-observance of the instructions for use) or due to interventions by unauthorized persons. The guarantee does not establish any claims for damages against us. The buyer's statutory claims for defects in accordance with § 437 BGB (German Civil Code) are not restricted. In case of claiming the warranty, the device must be sent to the manufacturer together with the original proof of purchase to:

LUXAMED GmbH & Co. KG
Daniel-Weil-Straße 3
D-89143 Blaubeuren
GERMANY

After-sales service:

Warranty and repair work must be carried out by trained and authorized personnel. The device must not be modified without the manufacturer's permission. Send the device carefully packed and sufficiently franked to your authorized dealer or directly to:

LUXAMED GmbH & Co. KG
Abteilung Service
Daniel-Weil-Straße 3
D-89143 Blaubeuren
GERMANY
Phone: + 49 (0) 7344 92905-0

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LUXAMED GmbH & Co. KG. KG.